


The Simple Secret To Dosing Capsules To Horses.....


 **WATER!** Just a little water before and after dosing is all it takes to administer a capsule in less than a minute!



Dose 10-20 ml of water using a dosing syringe without a needle *before and after* dosing capsules. This makes them slippery and easy to swallow



Place a filled capsule in the balling gun and insert into the horse's mouth. Gently push the balling gun up to the opening of the esophagus.

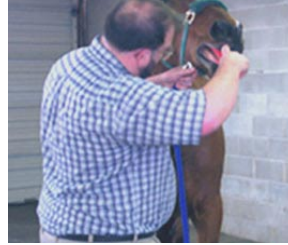

Push the balling gun's plunger to eject the capsule. Give more water to facilitate swallowing of the capsule.

- Dose materials in capsules that are not available in bolus form in less than 30 seconds without the worry of accidental aspiration as with nasal tubing.
 - Place any powder and most types of non-aqueous liquids and pastes into a capsule and insure that 100% of the dose is ingested by the target animal.
 - Capsules mask bad tasting or smelling materials, such as de-wormers, making them palatable.
 - Multiple tablets or smaller capsules can be filled into capsules to form a single dose.
- For more information visit www.torpac.com/horse.htm or call 1-973-244-1125**

